

# C.O.C. FARMACEUTICI SRL

ARTICOLO Foglietto Illustrativo	PRODOTTO Schema f.to 130 x 180 mm			
DIMENSIONI TOTALI 130 x 180 mm	PIEGATO	PASSO	CODICE 000000	SCALA 1.1
CARTA 000000000	GRAMMATURA gr. 0	TIPO PRODOTTO	F.TO BOBINA Ø 00 mm    Ø 000 mm	

Edizione	Data allestimento
0	09-12-2015

L'impianto stampa non deve assolutamente essere modificato o danneggiato

## VIZhial 0.4% Sodium Hyaluronate Eye Drops

VIZhial 0.4% is a sterile solution for ophthalmic use with a physiological pH, based upon a patented technology of cross-linking Hyaluronic Acid (HA) with Urea, therefore enhancing and improving the specific mechanical properties of native HA, resisting to hyaluronidase degradation much longer than native HA, thus with higher and prolonged lubricant and hydrating action than native HA. The hydrating web structure of cross-linked HA is capable to deliver water for a prolonged period improving the lubricating properties of native HA while boosting its moisturizing effects at the same time it is therefore indicated as an adjuvant in dry-eye syndrome. The cross-linked HA is also an efficient free radical scavenger, performing a re-epithelizing action for a higher protection of the corneal surface. The cross-linked HA provides to the product lubricating, moisturizing and repairing properties, capable of giving relief to ocular dryness due to dry eye syndrome, irritated or tired eye in case of inflammatory conditions or environmental factors.

VIZhial 0.4% relieves redness, dryness, fatigue and discomfort to the eyes and ocular mucosa caused by dry eye syndrome, environmental factors such as wind, smoke, pollution, prolonged exposure to sunlight, dried salt, sea water or pool water, or due to intense commitment to study, work, prolonged use of the computer, long period of driving, lacrimal abnormalities secondary to inflammations, eye surgery or eyelid aesthetic surgery and chronic blepharitis. Hydrates and lubricates the cornea and the conjunctiva in case of dry eye due to the use of contact lenses. The formula was developed to ensure maximum comfort already in the first moments after the application. It is suitable for everyday use, daily and night-time. It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

VIZhial 0.4% is indicated for:

- for reinstatement and improvement of tear film stability in dry eye syndrome;
- after refractive surgery;
- to relieve the eyes in cases of reddening, fatigue and discomfort of the eye or the ocular mucosa due to:
- environmental factors (air conditioned, excessive exposure to solar radiation, wind, smoke, pollution, sea water or pool water and dried salt);
- dryness of the eyes due to excessive fatigue caused by working on the computer for lengthy periods, study;
- lacrimal abnormalities secondary to inflammations, chronic blepharitis;
- lacrimal abnormalities secondary to eye surgery of various nature or eyelid aesthetic surgery;
- wearing soft or semirigid contact lenses.

### INGREDIENTS

Sodium hyaluronate crosspolymer aqueous solution (of which sodium hyaluronate crosspolymer 0,4%), Trometamol, Hydrochloridric acid 2N, Sodium chloride, W.F.I.. Preservative free.

### USAGE

- Cleanse your hands thoroughly;
- tear off the tamper-evident seal before the first administration;
- remove the protective cap and open the bottle;
- turn the bottle upside down and exert slight pressure on the bottle to instil 2 drops in each eye, unless otherwise prescribed;
- the gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions;
- after use close the bottle: the content should be re-used within the next 3 months from the first opening.

### WARNINGS AND PRECAUTIONS BEFORE USE

- Do not use the product if the container is damaged or not perfectly closed when you open it for the first time.
- Do not use the product in case of hypersensitivity to any ingredients of the formulation.
- Do not use the product after the expiry date indicated on the package and on the label.

### KEEP OUT OF THE SIGHT AND REACH OF CHILDREN

- The product is for ophthalmic use only.
- The product is intended for single-patient use.
- During application, do not touch the eye or any other surface with the container's dispensing edge.
- For its characteristic viscosity, immediately after instillation may cause a slight blur that quickly disappears.
- In case of problems during treatment interrupt the use and consult a doctor.
- Dispose of properly the container after use.
- Use within 3 months of opening.

### STORAGE

Store between 2°C and 30°C in a dry place away from heat and direct exposure to sunlight.

### PACKAGE


The package is composed of a bottle with polidose OSD locking system (ophthalmic squeeze dispensers), containing 10 ml of product free of preservatives.

### VALIDITY

With un-open pack, not exceed the expiry date stated on the box and on the label.



C.O.C. Farmaceutici S.r.l., Via Modena, 15,  
40019 - Sant'Agata Bolognese (Bo), Italy.

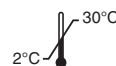

MEDICAL DEVICE

STERILE A

10ml

Last revision 01.01 04/2017

VISTO SI STAMPI

C.O.C. FARMACEUTICI SRL