

# EM PHARMA Cutter Guide

Leaflet

Flat size: 270 x 159mm Fold size: 23 x 159mm

50gsm

Fold Config: 270 Zig-Zag to 92mm in half x 2

# **FRONT**

159mm

23mm VIZhyal 0.2%

# **Sodium Hyaluronate 0.2% Eye Drops**

View Panel

# **Directions for Use**

- Ensure your hands are clean and dry before use.
- Shake bottle before use.
- Check the bottle seal is not broken or missing before use.
- To open the bottle: twist the cap anti-clockwise to break the seal.
- Tilt your head backwards and gently squeeze 1 to 2 drops into the eye.
- Blink several times to cleanse the eye.
- Repeat for the other eye as necessary.
- Replace cap after use.
- May be used while wearing contact lenses.
- Use as often as required/ safe for everyday use.

#### Warnings

- The product is sterile until first opened.
- Do not use if tamper-evident seal is missing or broken.
- Do not swallow the solution.
- If eye irritation occurs, discontinue use and consult an eye care practitioner. If condition worsens, consult a specialist.
- If you have been diagnosed with a medical eye condition or are using other eye medication, consult a healthcare professional before use.
- Do not use if allergic to any of the ingredients.
- May cause temporary blurring of vision; if affected do not drive or perform skilled tasks until vision is clear.
- For use in the eyes only.
- This product should not be used to soak or disinfect contact lenses.
- To avoid contamination, do not touch the dropper tip.
- Do not use after the expiry date.
- In the event of a serious incident that occurs during the use of this medical device, please report the incident to the manufacturer and the Competent Authority of the country in which you are located.
- Do not use if package is open or damaged.

## KEEP OUT OF THE SIGHT AND REACH OF CHILDREN

## Ingredients

Sodium Chloride 0.9%, Sodium Hyaluronate 0.2% in a Phosphate buffer.

## 184mm

92mm

**Storage**Store between 5°C to 30°C. Avoid direct sunlight.

Do not use if tamper evident seal is missing or damaged.

See base of bottle for LOT and EXP.

## **Shelf Life/After Opening**

35 months from DOM/ Use within 90 days of first opening.



East Midlands Pharma Ltd
Unit 1 Rothley Lodge,
Loughborough Road, Rothley,
Leicestershire, LE7 7NL, UK



Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta







270mm